


# Johne's Sample Collection and Shipping Procedure



## Sample Collection

### Serum Sample Collection Supplies:

1. Serum Separator tube (5ml)
2. Vacutainer or clean syringe with new needle for each sample.
3. Shipping container with ice packs
4. Sample submission form (Provided by API)

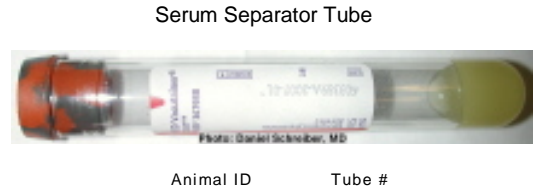


### Sample Preparation Procedure:

1. Write tube number and individual animal ID on tube.
2. Spin/centrifuge tubes prior to shipment.
3. Refrigerate or freeze prior to shipment.

## Sample Packaging

**Pack Johne's samples in separate labeled box.**

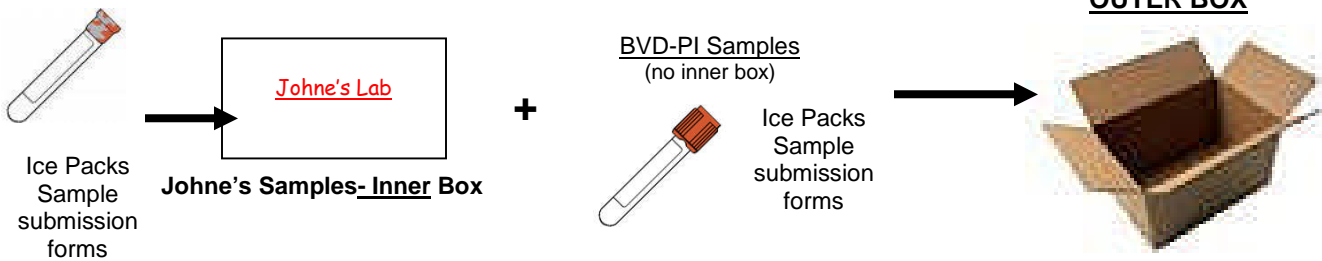

If shipping **Johne's samples only**, simply label outside of shipping box with "**JOHNE'S LAB**".

### Monday through Thursday

- o Place samples and ice pack in insulated envelope or bag prior to packing in box.
- o Samples should be wrapped tightly and sealed to avoid samples being loose in the box and to ensure samples remain cool during shipping.
- o Label or Write – "**JOHNE'S LAB**" on the exterior of the shipping box. (See above)

### Friday

- o Pack with extra ice packs to maintain samples over the weekend.

## Shipping

### REGULAR SHIPPING METHOD

#### UPS, FedEx, DHL, US Postal

Use overnight or 2 day services only.

If shipping two day only ship Mon – Wed

#### Send to:

Animal Profiling International, Inc  
6040 North Cutter Circle Suite 317  
Portland, OR 97217  
503-247-8066

1. Include client form with shipment
2. Sender will arrange payment

### FREQUENT SHIPPING METHOD

UPS (discounted overnight or 2 day rates)

**API will provide preprinted shipping labels**

Call for labels – (877) 278-1344

1. Include client form with shipment
2. Shipping will be billed on testing invoice

## Price

**Johne's Serum Screen -----\$5.95 per sample**

**For questions contact:** API Sales at (877) 278-1344 or API Lab at (503) 247-8066

**Headquarters:** 6040 N. Cutter Circle, Suite. 317, Portland, OR 97217 [www.animalprofiling.com](http://www.animalprofiling.com)